



Configurations: Audio mode



What is audio mode?

Audio Mode lets you choose where to focus your listening ability whether it's directly in front of you or around you.

Choose between these two options:

Frontal Mode

Helps you focus on conversations with someone directly, especially in noisy environments.



All-around Mode

Helps sounds and speech from all directions seem clearer, which is ideal for situations where you need to be aware of your surroundings.







How to adjust audio mode?

You can switch between modes in several ways: on your Glasses, with the Remote Control or on the Nuance Audio App.

On Nuance Audio Glasses
Double-tap the area on the right temple. An audio prompt will confirm your input.

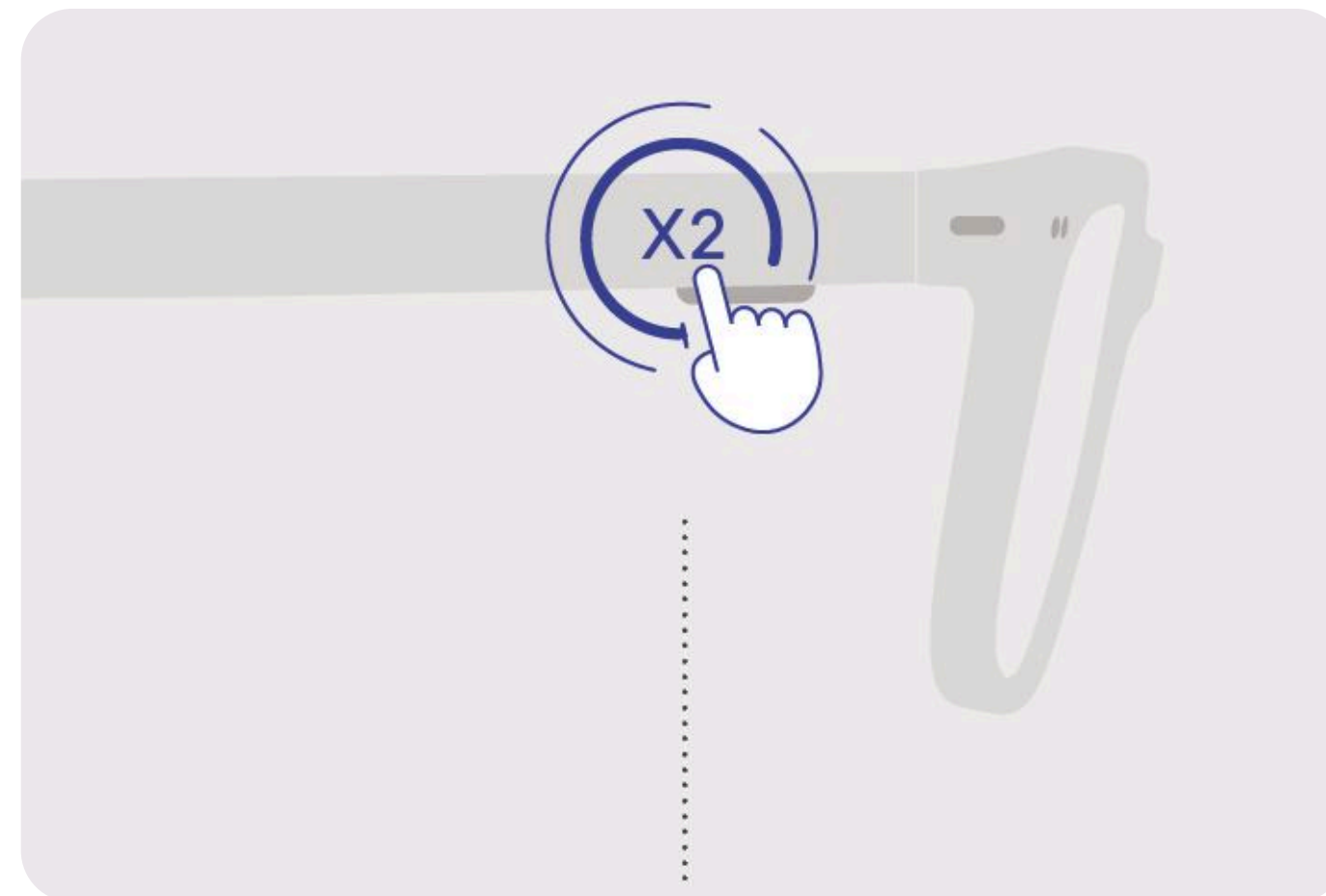


With Remote Control
Press the **Audio Mode button**.

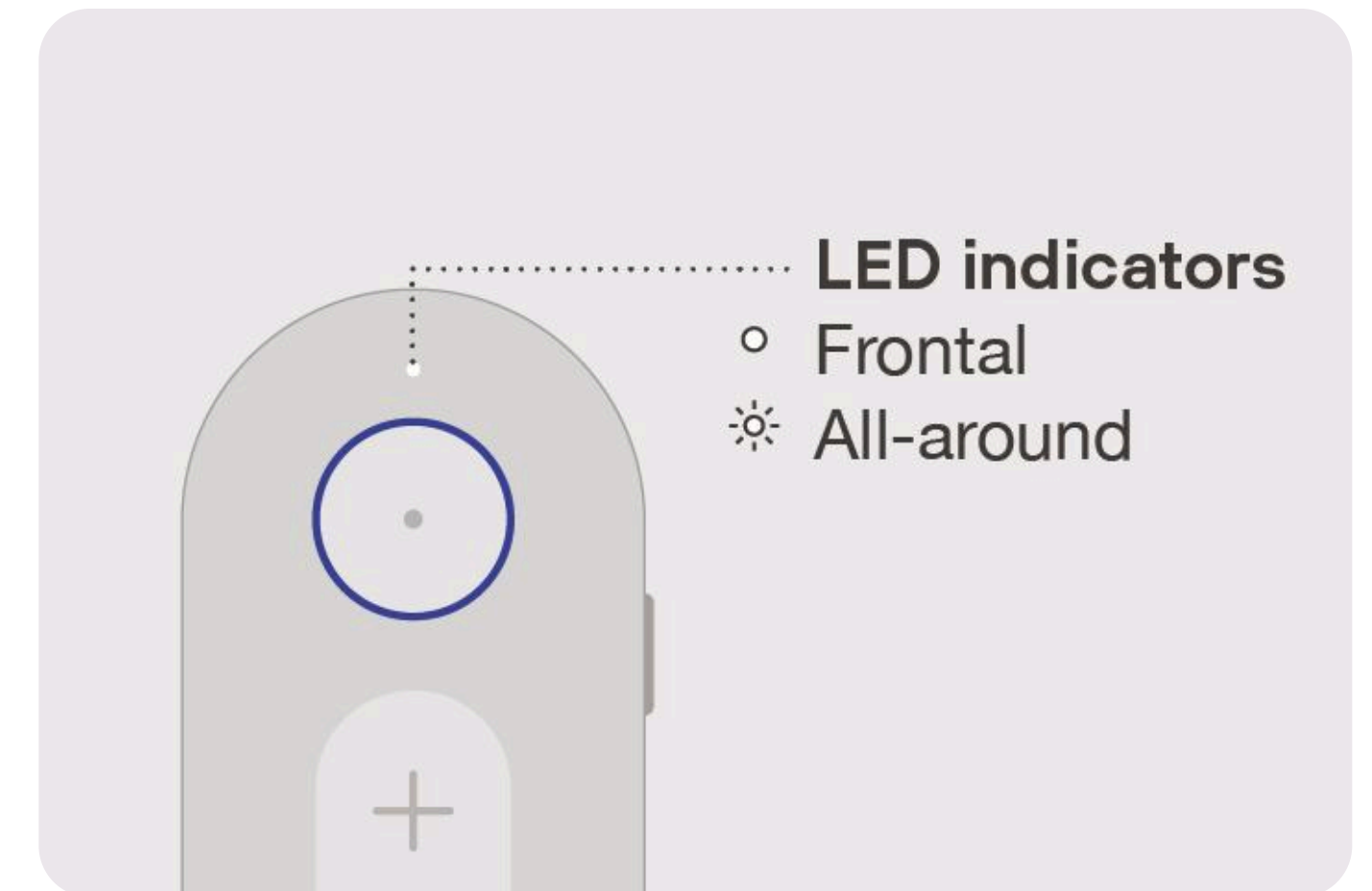



How to adjust audio mode?

With Nuance Audio App
Slide or tap on the **Audio mode toggle** available on the app's home page.

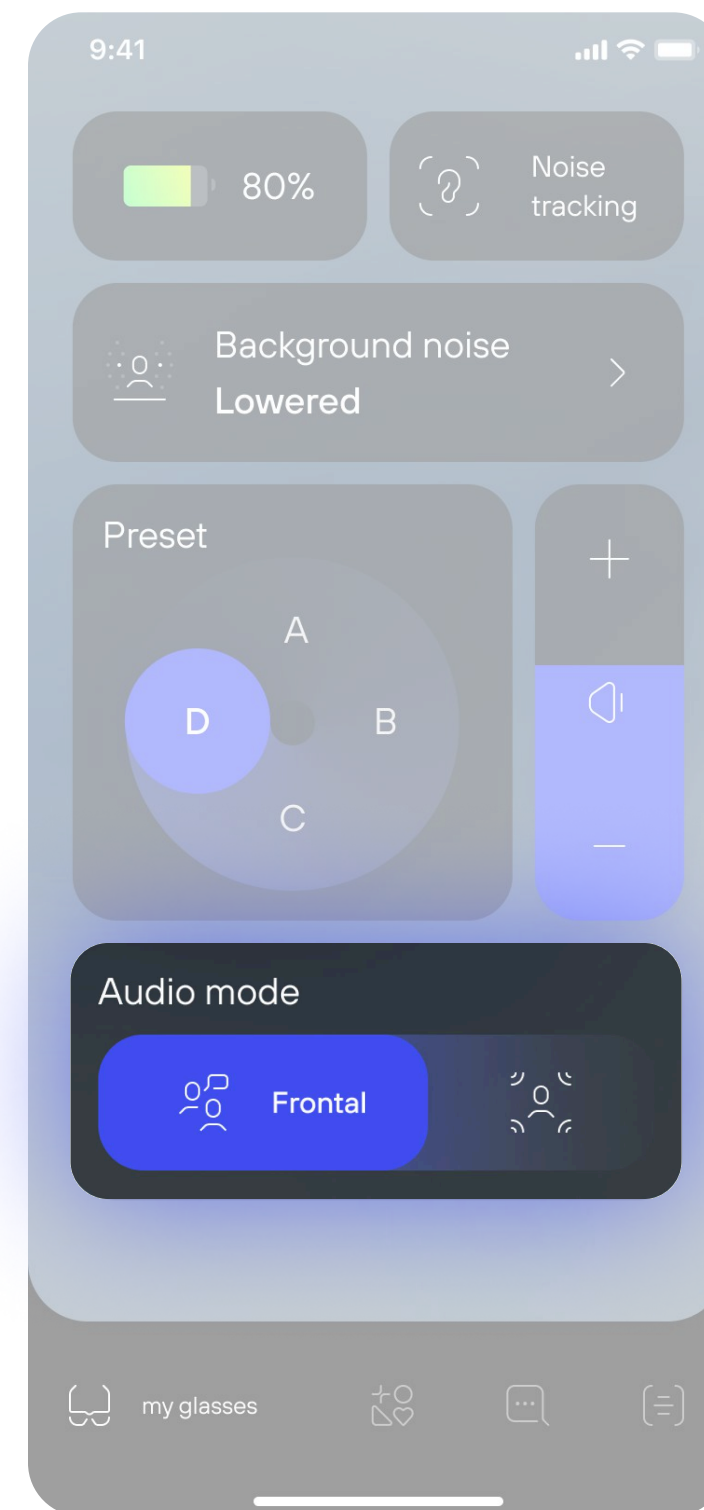


Frontal Mode
This mode helps you listen to sounds in front of you, making it easier to focus on conversations with someone directly.





How to adjust audio mode?

With Nuance Audio App
Slide or tap on the **Audio mode toggle** available on the app's home page.

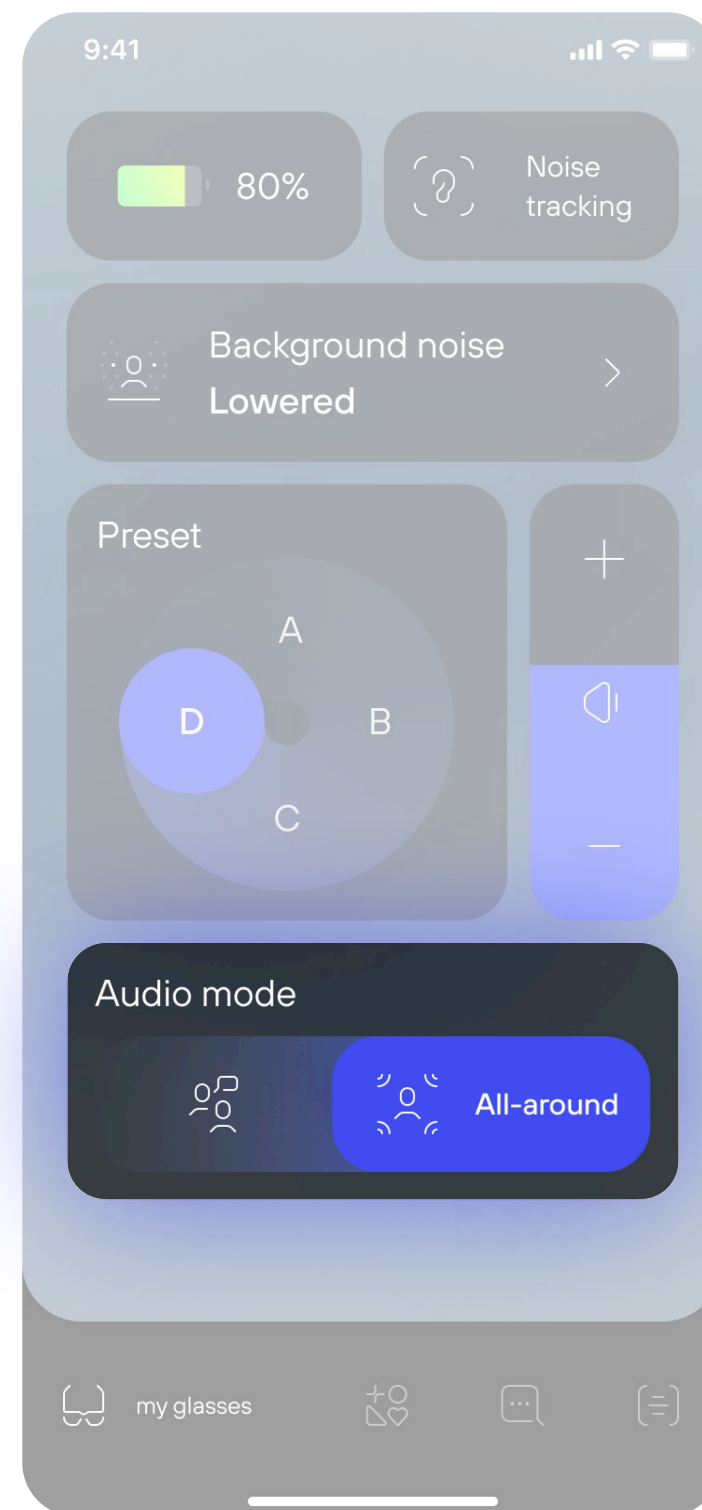

All-around Mode

This mode helps sounds and speech from all directions seem clearer, which is ideal for situations where you need to be aware of your surroundings.





For the most updated Instructions for Use
please follow this link

[↗ View Instruction for Use](#)